INSTRUCTIONS FOR USE

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JOBST® Relax

- | | | |
|--|--|--|
| EN Medical compression garments | NO Medisinsk trykbandasje | HU Gyógyászati kompressziós ruhadarab |
| DE Medizinische Kompressionsversorgung | DA Medicinsk s tøttestrømpe | RO Produs de compresie Medical |
| FR Vêtement de compression médical | FI Lääkinnällinen kompressiovaate | CZ Zdravotní Kompresní Výrobky |
| ES Prenda de compresión | IS Þrýstifatnaður í lækningaskyni | SL Medicinsko kompresijsko oblačilo |
| PT Peça de vestuário de compressão médica | ET Meditsiinilised kompressioonrõivad | RU Медицинская компрессионная одежда |
| IT Indumenti medicali compressivi | LT Medicininė kompresinė apranga | AR رداء ضاغط طبي |
| NL Medische compressiekous | LV Medicīniskās kompresijas apģērbs | TR Tibbi Kompresyon Giysisi |
| SV Medicinskt kompressionsplagg | PL Medyczna odzież uciskowa | HE בגד לחץ רפואי |

JOBST® RELAX PRODUCT FEATURES

(EN) PRODUCT FEATURES

(DE) PRODUKTEIGENSCHAFTEN

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(NL) PRODUCTKENMERKEN

(SV) PRODUKTEGENSKAPER

(NO) PRODUKTEGENSKAPER

(DA) PRODUKTFUNKTIONER

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(IS) VÖRUEIGINLEIKAR

(ET) TOOTE OMADUSED

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(PL) CHARAKTERYSTYKA PRODUKTU

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(RO) CARACTERISTICI ALE PRODUSELOR

(CZ) VLASTNOSTI VÝROBKU

(SL) LASTNOSTI IZDELKA

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Brand/CCL Marque/CCL	Polyamide (Nylon) Polyamide	Polyester Polyester	Elastane (Spandex) Élasthanne
JOBST® Relax/15-20 mmHg	72%	17%	11%
JOBST® Relax/20-30 mmHg	71%	17%	12%

- EN** Heat management promoted by yarn combination with monofilaments
- DE** Angenehmes Hautklima durch die Verwendung von speziellen Garnen und Monofilamenten
- FR** Fibre brevetée respirante associée à des monofilaments pour réguler la chaleur
- ES** Gestión del calor proporcionada por la combinación de hilos con monofilamentos
- PT** Superfície texturizada concebida para a micromassagem da pele e para apoiar a estimulação do fluxo linfático
- IT** Il filato combinato con monofilamenti promuove la gestione del calore
- NL** Ademend vermogen door speciale garens en tussenlaag met enkele vezels
- SV** Garnkombination med monofilament leder bort värme
- NO** Varmestyring forbedret via en garnkombinasjon med monofilamenter
- DA** Varmehåndtering fremmes med garn-kombinationen med monofilamenter
- FI** Strikket med en garnkombinasjon med monofilamenter som gir en forbedret regulering av varme
- IS** Hitastjórnun eflid með garni í samhæfingu við einþáttingu
- ET** Monofilamentidega lõng soodustab soojuse juhtimist
- LT** Tinkamos temperatūros palaikymas, kurį užtikrina verptų siūlų derinys su monofilamentais
- LV** Tekstildzija un monopavedieni nodrošina temperatūras regulāciju
- PL** Przędza monofilamentowa zapewnia efekt chłodzenia
- HU** A légáteresztő funkciót a monofilal kombinált fonal biztosítja
- RO** Gestionarea căldurii este ajutată de combinația de fire toarse și monofilament
- CZ** Řízení tepla podporováno kombinací přize s monofilamenty
- SL** Kombinacija preje z monofilamenti spodbuja upravljanje toplote
- RU** Пряжа с мононитьями обеспечивает терморегуляцию
- AR** إدارة الحرارة مدعومة بتوليفة من خيوط الغزل ذات الخيوط الأحادية
- TR** Monofilamanlı iplik kombinasyonu ile desteklenen isi yönetimi
- HE** בקרת חום שמושגת באמצעות שילוב חוטים וסיבים סינתטיים יחידים

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INSTRUCTIONS FOR USE

Dear Customer,

Thank you for choosing JOBST® Relax, a night compression garment that complements patients' recommended day-time lymphedema therapy while sleeping or relaxing. JOBST® Relax was designed to maintain edema reduction and counteract fluid accumulation at night.

1. Directions for use

Tips for wearing JOBST® Relax

The easiest way to don your JOBST® Relax garment is to always fold it and grab it double layered when pulling it up. This way, you have a better grip and avoid overstretching the material. Please refer to the pictograms shown on the insert card. If your garment has a zipper, open the zipper before and close it after donning.

JOBST® Relax for the arm

1. Fold the upper part over more than half of the garment.
 2. Pull the garment over your arm. The seam should be in line with your thumb to ensure the correct positioning of the garment.
 3. Unfold the upper part of the garment. If needed, flip it back down to be able to pull on the material double layered.
- The arm sleeve fits correctly when it covers the arm evenly without forming creases and when the seam is in line with the thumb. Check that the elbow is positioned correctly (see slim lines in elbow part).
 - To take off the arm sleeve first open the zipper (if present), fold the upper part downwards and gently pull on the hand/wrist.

JOBST® Relax for the leg

1. Fold the upper part over half of the garment.
 2. To don the garment, ensure that the heel pocket is facing downwards and then pull the garment with both hands over your foot all the way to the heel.
 3. Fold the lowest part of the garment back again until you see the heel section.
 4. Take hold of the folded edge of the garment and start pulling it over your leg, always holding the garment at the sides and never at the seam. Work the garment all the way up to the knee, smoothing the creases as you go up.
 5. Check the position of the knee and heel.
 6. For thigh high, flip over the upper part of the garment and back to pull it double layered up your thigh. Stretch the last part evenly until it ends below your crotch.
 7. Close the zipper (if present) and adjust toe and heel areas for comfortable fit.
- The leg sleeve fits correctly when it covers the leg evenly without forming creases and the seam runs straight in the center back of the leg.

For knee high, the garment should end 1-2 cm below the knee bend.

- To take off the leg sleeve first open the zipper (if present), then fold the upper part downwards and back and pull gently on the double layered material.

Tip: A shoehorn with round edges may be of help when doffing the garment. Pull the garment down as much as possible, slide the shoehorn between the fabric and the heel and then stretch it over your heel.

Special notes:

- Caution: long fingernails and jewelry may damage the garment when putting it on.
- For easier application use rubber gloves or a donning aid.
- Do not bring the compression garment into contact with greasy ointments, body lotions or similar substances.
- If you like to use moisturizer, e.g. Cutimed Urea 10%, allow the moisturizer to be absorbed completely before you put on your JOBST® Relax and wash the garment frequently to avoid dirt /moisturizer film build up on the surface.
- Loose threads or loops should not be cut off, since this may cause holes or tears to form in the garment.
- Intended to be used by a single patient only.

2. Care instructions

JOBST® compression garments should, if possible, be washed daily, or at least every other day.

When washing, turn the garment inside out and close the zipper, if present.

For this purpose, you may use a regular mild detergent. Do not use laundry additives such as fabric softener, optical brightener or stain remover. Wash with similar colours only. Spin at low rate (600 cycles), dry at low temperature in a dryer or lay flat to dry.

Do not wring. Do not dry the compression garment on a radiator or in direct sunlight.

3. Intended use and indications

JOBST® Relax is indicated as a complementary nighttime compression garment to recommended daytime therapy for patients suffering from primary or secondary lymphedema and/or other forms of edema at night and during relaxation.

JOBST® Relax is indicated once the volume and size of the affected limb has been reduced (i.e. during the maintenance phase), to allow correct fit of the garment.

JOBST® Relax helps to maintain the reduced swelling and shape of the limb and counteracts further fluid accumulation during the night.

Please refer to the packaging label for the compression level. The required compression class must be determined by your physician.

4. Contraindications

JOBST® Relax should not be used in case of:

- Arterial insufficiency
- Deep vein thrombosis (DVT)
- Untreated / decompensated congestive heart failure, acute cardiac failure
- Untreated cancer
- Very deep skin folds
- Untreated septic phlebitis
- Phlegmasia coerulea dolens
- Absent or severely impaired sensation, severe peripheral neuropathy
- Known sensitivity to any of the product components
- The variant with zipper is not indicated for overweight patients with fragile skin or with medical conditions that affect blood supply, making skin more fragile or causing movement problems.
- Open wounds not covered with an appropriate dressing

5. Cautions

- Weeping dermatoses
- Untreated localized skin infection
- Immobility (confinement to bed with no ability of active movement).
- Impaired sensitivity of the limb, peripheral nerve damage.
- Conditions in which increased venous and lymphatic return is not desired. Please follow your healthcare professional's recommendations.
- Appropriate hygiene and skin care should be applied.
- JOBST® Relax compression garments should only be used after consultation with your healthcare professional and be checked regularly by them on the appropriateness of your therapy, also considering age and health status.
- The healthcare professional decides whether compression in general can be used in patients suffering from the following diseases: rheumatoid arthritis; complex regional pain syndrome (CRPS, M. Sudeck); malignant lymphedema; gangrene.
- Patient measurements should be taken by a trained person to ensure correct product fit.
- Ill-fitting compression garments may cause constriction leading to e.g. skin necrosis, pressure ulcers and pressure related peripheral nerve damage.
- Open wounds should be covered with an appropriate dressing before the compression garment is donned.

- If you notice discomfort, pain, numbness, changes in skin color, appearance of new wounds or swelling during wear, stop using the product and contact your healthcare professional.
- Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent state authority.
- When re-ordering a garment after 6 months we recommend remeasuring the patient as the limb circumference might have changed.

6. Direction for storage / guarantee

JOBST® Relax is intended for immediate use and no expiry date for storage is indicated on these. Due to wear or slackening of the material, the medical efficacy can be guaranteed for a wearing period of six months at most.

This assumes correct handling (e.g. product care, donning and doffing).

Please protect your compression garments from direct exposure to sunlight, heat, and moisture.

We recommend storage at room temperature in the provided storage bag.

If you have questions, please contact your healthcare professional or JOBST® brand supplier.

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BSN-JOBST GmbH
Essity UK Ltd, Southfields Road, DUNSTABLE, Beds, LU6 3EJ
Australian Distributor: BSN medical (Aust.) Pty Ltd